



Altadonics Pearls: 3 Mould Must Do's

- **CLEAN & DRY** denture completely of any materia alba & dry BEFORE making impression! If you see moisture in the denture after alginate pour, use the air vac to dry it out BEFORE making the second pour! We suggest sanitizing in your ultrasonic for 5-7 min, then using a stiff brush to remove any remaining residue.
- **SEPARATOR ON GREEN RUBBER ONLY.** Never apply separator to the denture!
- **SNAP** the lid completely shut and check to make sure it is secure.

Reminders for Staff

- Moisture, separator and materia alba all cause distortion. If these are present, the rubber mould **will** record it.
- When lid is **not** properly sealed, discrepancies occur

(The above is intended for those responsible for making the Altadonics Denture Moulds.)